Gebrauchsanweisung | User Manual

Bracket AED 3 for BLS Bag

P/N: 1042100004

■ INHALTSVERZEICHNIS | TABLE OF CONTENTS

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■ WELCOME | SAFETY INSTRUCTIONS

Welcome

Thank you for purchasing the Bracket AED 3 for BLS Bag.

In this user manual you will find technical information, instructions for use, and mounting directions.

Safety Instructions

- Read and understand the instructions before installing or operating the Bracket AED 3 for BLS Bag.
- Follow all safety instructions and save this user manual in a safe place for future use.
- The manufacturer accepts no liability for damage caused by improper operation, mounting, or assembly.
- Repair and maintenance of the Bracket AED 3 for BLS Bag can only be performed by authorized personnel. Please consult directly to CRS medical GmbH Technical Support if the device is defective or damaged. Any changes to the device or its accessories which are not performed by CRS medical GmbH will void all product warranties. CRS medical GmbH will not be liable for unauthorized repair or maintenance.
- Check the Bracket AED 3 for BLS Bag and do not use if:
 - there is visible mechanical damage on the locking mechanism
 - there is visible mechanical damage on the holder

■ APPLICATION

The Bracket AED 3 for BLS Bag is intended to be installed in ambulances which use the ZOLL AED 3™ defibrillator in combination with ZOL medical's standard carry case. Using a different protective bag is not allowed.

The holder may only be used in a protected environment. These include massive walls in the interior of buildings and stable walls of EMS ground vehicles.

Please also follow all manufacturer's instructions for the defibrillator.

■ SCOPE OF DELIVERY

1 Bracket AED 3 for BLS Bag (P/N: 1042100004)

1 User Manual

■ TECHNICAL DATA

Weight	3600g
Dimensions (HxWxD)	351,5x258x183 mm
Operating temperature	0°C to +40°C, 5% to 95% Humidity, non-condensing
Storage temperature	-30°C to +70°C, 5% to 95% HR, non-condensing


In the pursuit of product improvement, we reserve the right to make visual and/or technical changes to the product.

■ ACCESSORIES

There are no additional accessories available.

■ INSTALLATION

Install the Bracket AED 3 for BLS Bag with 8 screws size M6 according to the drilling diagram below.

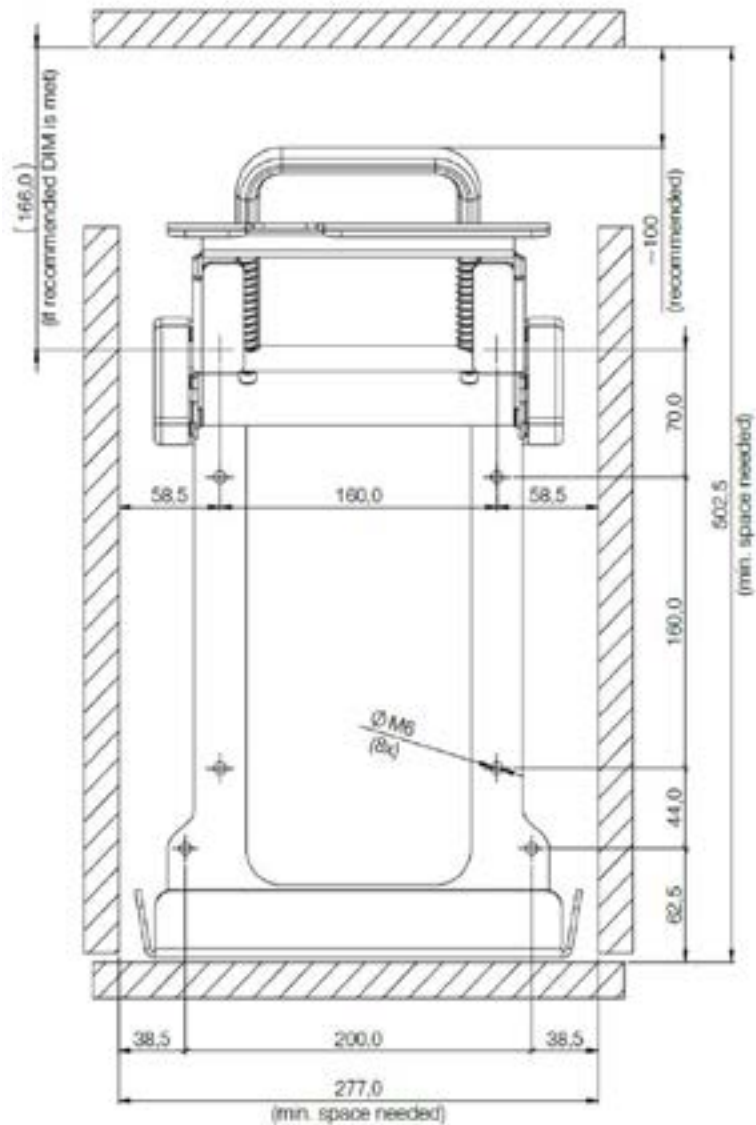(All measurements are in mm)

■ OPERATION

Opening

Unlock the Bracket AED 3 for BLS Bag by pulling the red handle forward (1) and then move it upwards (2).

The locking device rests in the upper position (3) so the device can be removed.

Closing

Make sure that the ZOLL AED 3™ with the rubber feet of the standard carry case is flat against the back of the Bracket AED 3 for BLS Bag.

To close the Bracket AED 3 for BLS Bag there is no unlocking, simply grab the red handle and pull the cover down. The Bracket AED 3 for BLS Bag locks when the handle is released and then gently pushed down. An audible click should be heard.

■ CLEANING | DISINFECTION

Please remove the ZOLL AED 3™ out of the bracket before cleaning.

The Bracket AED 3 for BLS Bag can be cleaned with a damp, lint-free cloth and commercially available mild plastic cleaners. Please ensure that the cleaner does not contain any solvents.

Commercially available disinfectants can be used for disinfection.

■ SYMBOLS

See user manual/note safety instructions

Reference

Lot number

Manufacturer

■ DISPOSAL

Packing is raw material and therefore recyclable. On behalf of environment protection please dispose the packing sorted. Dispose and recycle all packaging according to your local laws and regulations

For more detailed information regarding environmentally friendly waste disposal please contact the manufacturer.

■ SERVICE | MANUFACTURER

CRS medical GmbH
Loherstraße 6
D - 35614 Asslar

T: +49 6441 38331-0
F: +49 6441 38331-29
info@crs-medical.com
www.crs-medical.com

□ Loherstraße 6
□ D - 35614 Asslar

□ T: +49 6441 38331-0
□ F: +49 6441 38331-29

□ info@crs-medical.com
□ www.crs-medical.com