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■ WELCOME | SAFETY INSTRUCTIONS

Welcome

Thank you for purchasing the Bracket D500.

In this user manual you will find technical information, instructions for use, and mounting directions.

Safety Instructions

- Read and understand the instructions before installing or operating the Bracket D500.
- Follow all safety instructions and save this user manual in a safe place for future use.
- The manufacturer accepts no liability for damage caused by improper operation, mounting, or assembly.
- Repair and maintenance of the Bracket D500 can only be performed by authorized personnel. Please consult directly to CRS medical GmbH Technical Support if the device is defective or damaged. Any changes to the device or its accessories which are not performed by CRS medical GmbH will void all product warranties. CRS medical GmbH will not be liable for unauthorized repair or maintenance.
- Check the Bracket D500 and do not use if:
 - there is visible mechanical damage on the locking mechanism
 - there is visible mechanical damage on the holder

■ APPLICATION

The Bracket D500 is intended to be installed in ambulances which use the Mediana D500 defibrillator in combination with the original bag. Due to safety requirements it is not possible to use the rear bag of the device.

The holder may only be used in a protected environment. These include massive walls in the interior of buildings and stable walls of EMS ground vehicles.

Please also follow all manufacturer's instructions for the defibrillator.

■ SCOPE OF DELIVERY

1 Bracket D500 (P/N: 1041100030)

- Consisting of:
 - D500 Adaption w/o power (P/N: 1042100030)
 - D500 Wall Fastening w/o power (P/N: 1042100031)

1 User Manual

■ TECHNICAL DATA

Weight	7500 g
Dimensions (HxWxD)	486,5x295x256,5 mm
Operating temperature	0°C to +40°C, 5% to 95% Humidity, non-condensing
Storage temperature	-30°C to +70°C, 5% to 95% HR, non-condensing


In the pursuit of product improvement, we reserve the right to make visual and/or technical changes to the product.

■ ACCESSORIES

There are no additional accessories available.

■ INSTALLATION

Install the Bracket D500 with 12 screws size M6 according to the drilling diagram below.

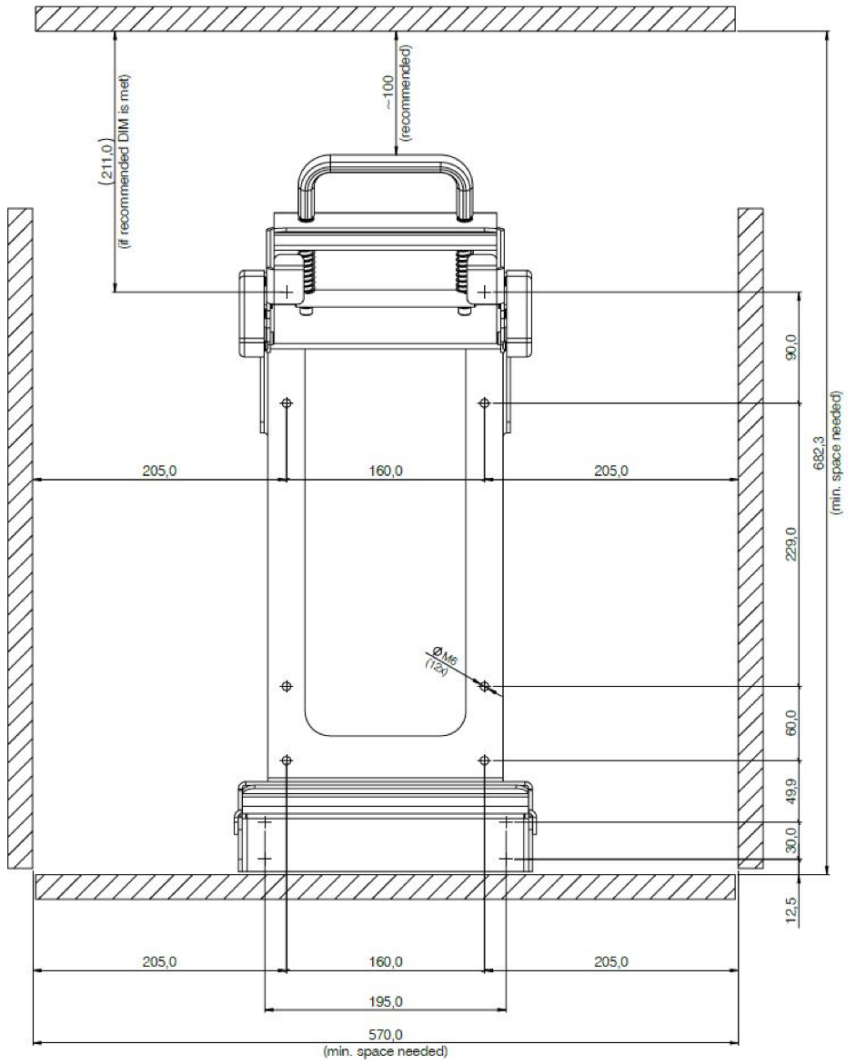(All measurements are in mm)

Remove the device's rubber feet. Keep them safe in case you want to use the device without bag and bracket some day.


Insert the Ø15,5mm plastic washers into the cavity of the rubber feet.

Now put the bag with its lower strap over the bottom of the device.


Mount the D500 Adaption w/o power using the higher Spacers in the rear and the flat ones in the front of the device. Position everything properly and fix it using the washers and screws delivered with the Adaption.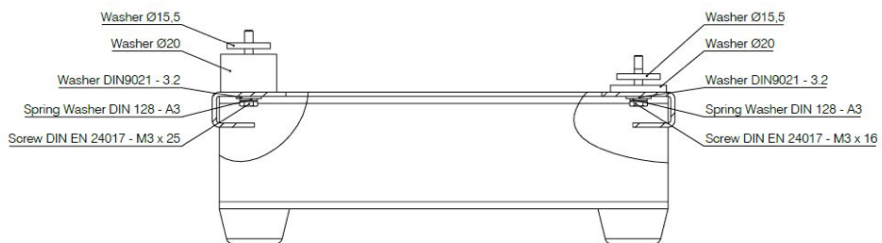

■ OPERATION

Opening

Unlock the Bracket D500 by pulling the red handle forward (1) and then move it upwards (2).

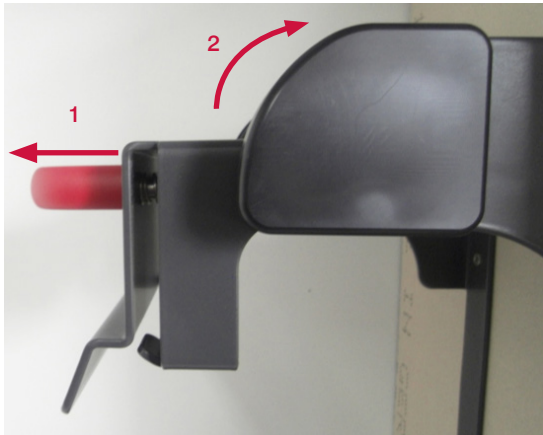The locking device rests in the upper position (3) so the device can be removed.

Closing

Make sure that the Mediana D500 is flat against the bottom of the Bracket D500 with the D500 Adaption power w/o power.

To close the Bracket D500 there is no unlocking necessary. Simply grab the red handle and pull the cover down. The Bracket D500 locks when the handle is released and then gently pushed down. An audible click should be heard.

■ CLEANING | DISINFECTION

Please remove the Mediana D500 out of the bracket before cleaning.

The Bracket D500 can be cleaned with a damp, lint-free cloth and commercially available mild plastic cleaners. Please ensure that the cleanser does not contain any solvents.

Commercially available disinfectants can be used for disinfection.

■ SYMBOLS

See user manual / note safety instructions

CE-Declaration of conformity according to EU-directive 93/42/EEC by manufacturer.

Reference

Lot number

Manufacturer

■ DISPOSAL

Packing is raw material and therefore recyclable. On behalf of environment protection please dispose the packing sorted. Dispose and recycle all packaging according to your local laws and regulations.

For more detailed information on nonpolluting disposal please contact service.

■ SERVICE | MANUFACTURER

CRS medical GmbH
Loherstraße 6
D - 35614 Asslar

T: +49 6441 38331-0
F: +49 6441 38331-29
info@crs-medical.com
www.crs-medical.com

□ Loherstraße 6
□ D - 35614 Asslar

□ T: +49 6441 38331-0
□ F: +49 6441 38331-29

□ info@crs-medical.com
□ www.crs-medical.com