Gebrauchsanweisung | User Manual
Single Charger
P/N: 1042100404 (P/N: 2103733001)

■ INHALTSVERZEICHNIS | TABLE OF CONTENTS

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■ WELCOME | SAFETY INSTRUCTIONS

Welcome

Thank you for buying the Single Charger.

To ensure safe operation we advise you to get familiar with this user manual. The producer does not assume liability for damage or failure due to incorrect application or mounting.

Please read this manual carefully before commissioning.

Keep this user manual in safe custody for future reference and allow for safety instructions.

Safety Instructions

- Please read this manual carefully before use.
- The Single Charger is exclusively intended for use with ZOLL® lead accumulators and CRS PowerDrive.
- Should the device have been damaged by a blow, crash or accident, or should liquid have penetrated its electronics the charger must not be used again. Please contact customer services.
- This charger is not suitable for sterilization or for use with an autoclave.
- Do not allow that cleanser or liquids enter the openings.

■ APPLICATION

The Single Charger is exclusively intended for use with ZOLL® lead accumulators and CRS PowerDrive.

■ SCOPE OF DELIVERY

Single Charger (P/N: 1042100404)

Consists of:

- 1 Charger incl. adapter
- 1 Power cable
- 1 User manual

■ TECHNICAL DATA

Weight	800 g
Dimensions (HxWxD)	70x115x60 mm
Operating temperature	0°C to +40°C, 5% to 95% HR, non-condensing
Storage temperature	-30°C to +70°C, 5% to 95% HR, non-condensing

Specifications subject to change without notice.

■ ACCESSORIES

No accessories available.

■ APPLIANCE

Connection to power

To connect the charger to power use the provided power cable 230V/50Hz.

Connection to battery

Plug the adapter with its pins to battery.

Indication of charge state

The charger has two LED's indicating status as follows:

Green: the charger is connected to power.

Orange: a poor battery is connected to charger.

If the orange LED flashes only once it shows that an already loaded battery is connected.

The orange LED will be on during the duration of charging.

To indicate the battery is fully charged the orange LED will turn off.

■ CLEANING | DISINFECTION

For cleansing and disinfection please use mild standard cleaner and disinfectants and make sure they do not affect plastic surfaces.

■ SYMBOLS

See accompanying documents/ user manual

Applicable for the European Economical Area: CE-Declaration of conformity according to EU-directive 93/42/EEC by manufacturer

Protection class II

Reference

Serial number

Manufacturer

■ DISPOSAL

Packing is raw material and therefore recyclable. On behalf of environmental protection please sort packing before disposing. dispose packing sorted.

This product is subject to directive 2002/96/EC and consequently has to be separated from general household garbage. Dispose and recycle this product according to your local laws and regulations.

For more detailed information on proper disposal please contact service.

■ SERVICE | MANUFACTURER

CRS medical GmbH
Loherstraße 6
D - 35614 Asslar

T: +49 6441 38331-0
F: +49 6441 38331-29
info@crs-medical.com
www.crs-medical.com

□ Loherstraße 6
□ D - 35614 Asslar

□ T: +49 6441 38331-0
□ F: +49 6441 38331-29

□ info@crs-medical.com
□ www.crs-medical.com